


Setting up Gaumard Vitals in Control Room with UNI running on Separate PC

Justin Fedoryk - 2022-09-22 - High-Fidelity


Description: Process for connecting Gaumard Vitals in control room to display in sim room, with PC controlling Gaumard Manikin is in sim room or another room.

Equipment: Hal, Noelle, Victoria, Universal computers
9-205/9-209/9-211/9-215/9-209/9-210


Software: Uni, Gaumard Vitals

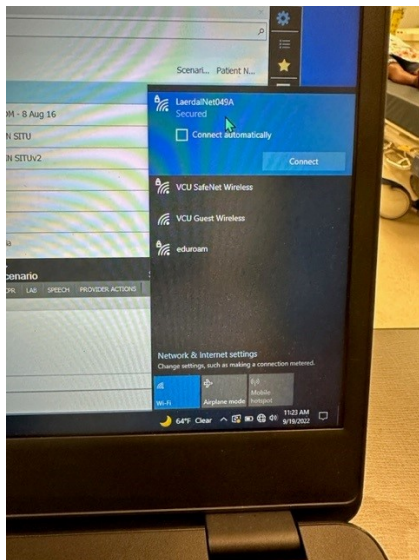

1. Connect Universal PC in control room (Monitor PC) and PC that will be connected to and control Hal, to a common WIFI signal. (in this example, we are using a Laerdal Router used for SimMan3G)

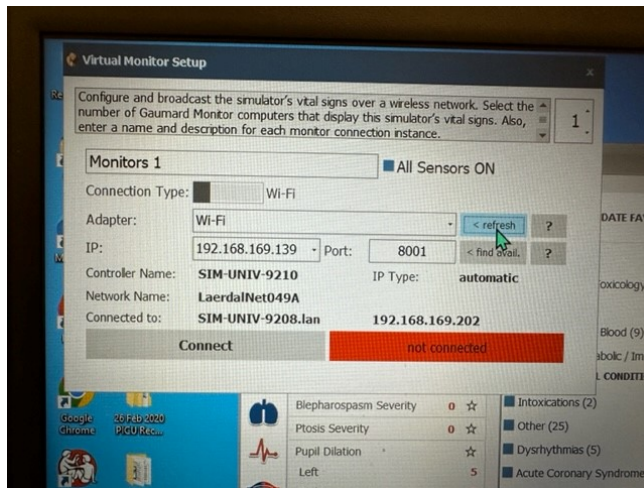

2. On PC controlling manikin, open Monitor Configuration Settings, and select "Refresh" if wifi option is not showing

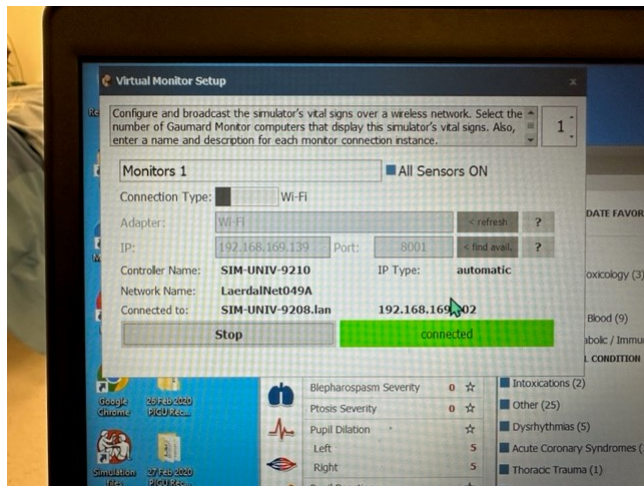

3. and Connect
- Note the Controller Name



4. Enter the “Controller Name” that was noted from step 3 (in this case it is SIM-UNI-9208) and make sure “Connection Type” Is WIFI. Then select “Connect”

You may have to add .LAN after the controller name i.e. SIM-UNI-9208.LAN